

## Presto LCD ERROR CODES

### General

Please note: Never try to resolve a fault yourself. Contact the dealer for service.

### Error codes

1. An error code may show on the display in a abnormal situation.
2. Refer to the table below for the error codes, their meaning & possible resolution
3. If resetting does not help, contact the dealer to resolve the error.
4. Report the error message to your dealer.

Error Code	Fault	Possible Resolution
1	Battery almost empty	The batteries are almost empty or over-discharged • Charge the batteries
2	Low battery fault	The batteries are empty. • Charge the batteries. • Check the battery and its connections and wiring.
3	Battery high fault	Battery voltage is too high. Can occur when the battery is overcharged and/or when driving down a long slope. • Reduce speed when driving downhill to avoid excessive regenerative charging.
4	Time-out current or controller overheated	The engine has been overloaded for too long. • The mobility scooter may have come to a stop. Turn off the controller, wait a few minutes, and then turn it back on. • There may be an engine fault. • Check the motor and its connections and wiring.
5	Parking brake malfunction	The parking brake release switch has been activated or the parking brake is defective. • Check the parking brake and its connections and wiring. • Make sure that the corresponding switches are in the correct position.
6	Driving restriction	A stop function is active, a limitation has occurred during charging, or an OONAPU (OONAPU: Out Of Neutral At Power Up) situation has occurred. • Remove the stop situation (seat raised, etc.) • Disconnect the battery charger • Make sure the throttle is in the neutral position when the controller is turned on.
7	Storing in potentiometer	The throttle may need to be recalibrated. There may be a problem with the throttle lever, potentiometer, SRW or associated wiring. • Check the battery and its connections and wiring.
8	Motor voltage fault	There is a fault in the motor or its associated wiring. • Check the motor and its connections and wiring.
9	Other	There may be an internal fault in the controller. • Check all connections and wiring.